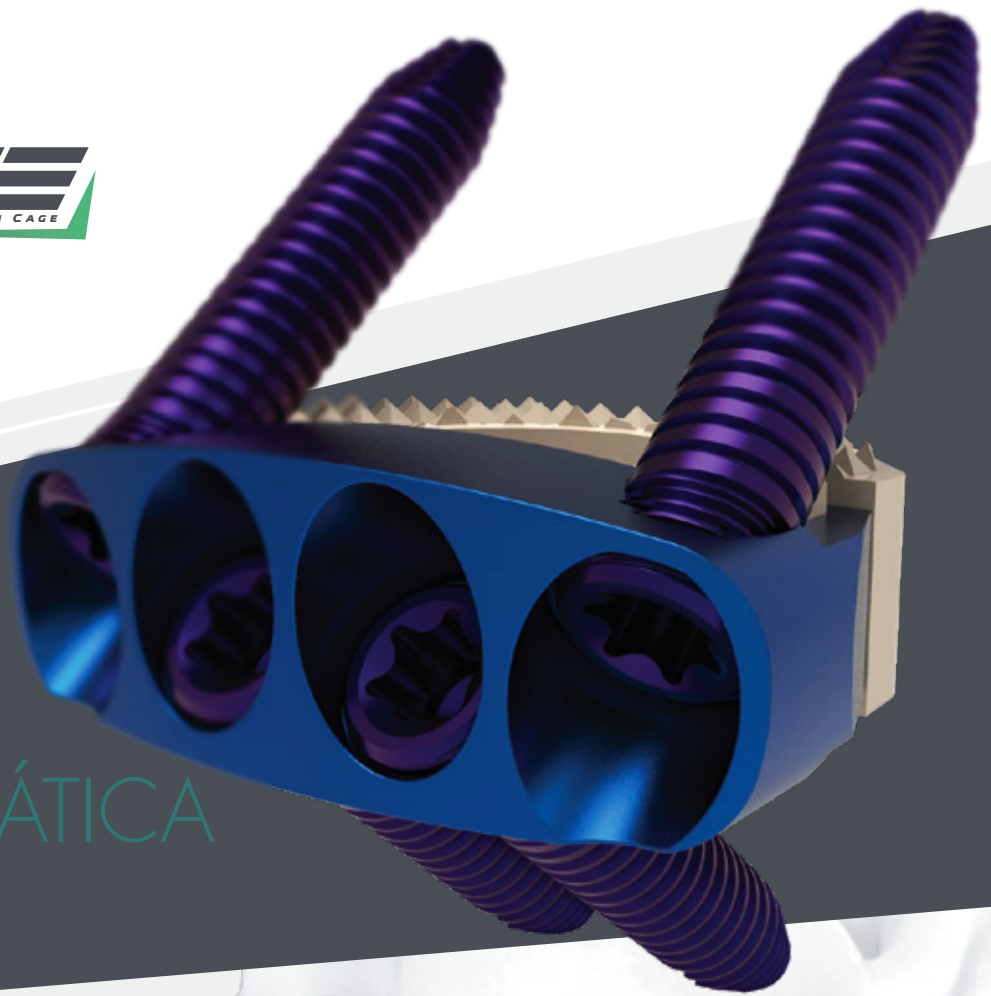
SAI[®]
SOUTH AMERICA IMPLANTS

TÉCNICA QUIRÚRGICA

SURGICAL TECHNIQUE

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X-Zone System - Cervical intersomatic fusion locking cage

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X-ZONE is a stand-alone device for cervical intersomatic fusion. It combines the functionality of an intersomatic spacer with the benefits of an anterior cervical locking plate. The spacer is made of PEEK, while the fixation section is entirely made of titanium, which enables the device to secure firmly to the vertebral body.

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This description of the surgical technique is not enough for immediate use in clinical application. Hands-on training with an experienced surgeon is highly recommended.

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With the standard surgical approach, expose the vertebral bodies to be fused. Prepare the fusion site according to the appropriate technique for the indication given.

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Place the patient in the supine position on the radiolucent operating table. The patient's neck should be placed in a sagittal neutral position, resting on a pillow. If segment C6-C7 is going to be operated on, check that the patient's shoulders do not block the radioscopic view. In any case, both vertebrae should remain fully visible.

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Spot the segment affected using radioscopic X-ray and make the incision.

Using the anterior approach to the cervical spine, expose the vertebral disc affected and the adjacent vertebral discs.

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It is essential to place the distractor carefully to prevent any damage to the soft parts.

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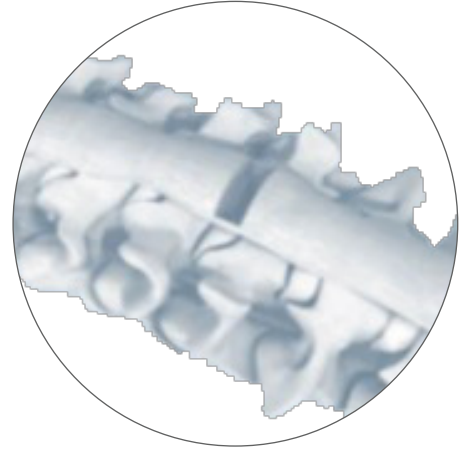
Prepare the spondylosyndesis using the proper technique based on the specific instruction.

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It is essential to distract the segment to restore disc height and provide the proper approach to the intervertebral space.

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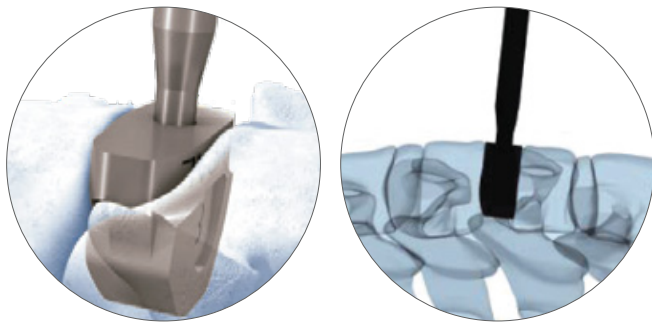
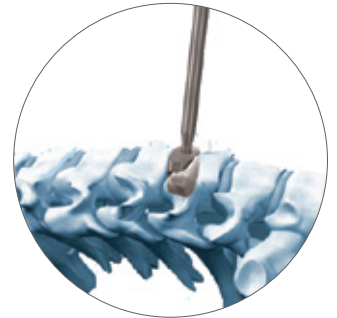
With the standard surgical approach, expose the vertebral bodies to be fused. Prepare the fusion site according to the appropriate technique for the given indication.

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Choose a trial spacer of the appropriate height and depth depending on the height of the intervertebral space, the preparation technique and patient's anatomical characteristics.

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Place the trial spacer with the correct cranial/caudal alignment and insert it carefully into the intervertebral space.

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Trial spacers do not have a depth limiter, therefore an image intensifier should be used to verify the position throughout insertion. Once the segment is fully distended, the trial spacer must fit tightly and accurately between the endplates. Choose the appropriate size for the implant based on each patient's anatomical characteristics; otherwise, it can cause harm to the patient.

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The mallet can be used to help insert and/or remove the trial spacer. If preferred, a larger coupling can be attached to the trial spacer.

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Height of the trial spacer is 0.8 mm less than that of the corresponding implant which should be taken into account for teeth penetration into the vertebral end plate. Trial spacers should not be used for final implantation and must be removed before inserting the X-Zone locked cervical cage.

To minimize patient risk, it is recommended:

- Try using a trial spacer with lower height before using a trial spacer with higher height; and
- To use standard size trial spacers rather than using large size trial spacers.

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Step 03

2026-000 RELLENO ÓSEO DE HIDROXIAPATITA BOVINA GRANULOS ESPONJOSOS 2,5 cc.

2026-000 BOVINE HYDROXYAPATITE BONE PACK WITH CANCELLOUS PELLETS 2.5 cc.

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Place the appropriate X-Zone locked cervical cage implant into the packing block.

Use the cancellous bone impactor to firmly pack graft material into the implant cavity.

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Use the aiming device for X-Zone locked cervical cage or implant holder to insert the implant into the interdisc space. It is recommended to place medial screws caudally.

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Use of the aiming device for the X-Zone locked cervical cage

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Attach the X-Zone locked cervical cage aiming device to the implant by aligning the implant screw holes with the retention features on the aiming device and then expand the aiming device. If required, tap the top of the aiming device with the mallet to advance the implant into the intervertebral space.

If necessary, tap the top of the aiming device with the mallet to advance the implant into the intervertebral space. If distraction has been applied, release and leave the aiming device attached to the implant.

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Alternatively, the implant can be inserted into the intervertebral space with the forceps-type implant holder. Once the implant has been partially inserted into the disc space, it can advance to the correct posterior depth using the flat impactor and / or the impactor with the ball tip.

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Verify the final position of the implant in relation to the vertebral bodies in the AP and lateral direction by means of intraoperative images. The X-Zone locked cervical cage has a posterior X-ray marker attached to the implant to allow for intraoperative accurate X-ray assessment of the implant position.

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Implants can be oriented in any direction (medial screws pointing cranially or caudally). To facilitate the placement of the device, the first thing to do is to orient the implant with the medial screws pointing caudally. If the placement is compromised, place the implant in the other direction (with medial screws pointing cranially).

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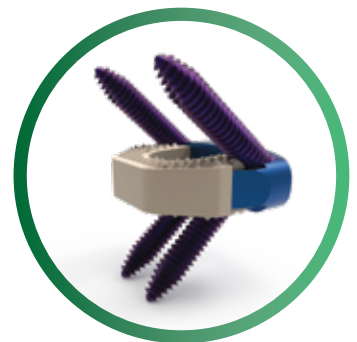
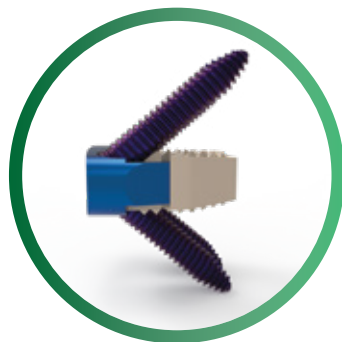
Fixation using the aiming device for the X-Zone locked cervical cage

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The X-Zone locked cervical cage aiming device allows to insert a screw with the instrument attached to the implant. This helps to keep the implant in place while the other holes are prepared and the screws are inserted.

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Paso 05.A.1
Step 05.A.1

Drill the first pilot hole with the drill bit and screw it into the hole of the guide

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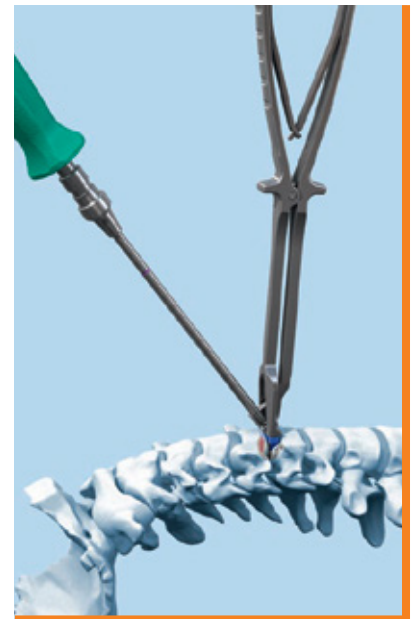
Select a drill bit with a suitable depth stop. Insert the drill bit into the handle and into the screw hole in the aiming device and drill until the bit stop contacts the guide.

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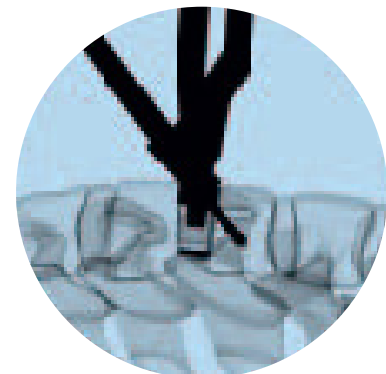
Use intraoperative images to verify the position of the drill bit. Remove the drill bit.

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When using the drill bit along with the aiming device, take care and apply only axial forces to the drill bit. The bending forces applied when the tip of the drill bit is engaged in the aiming device can break it and potentially increase patient risk.

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*Select the appropriate screw length according to the preoperative planning and intraoperative results.
Assemble the torque limiter to the screwdriver and handle.*

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Use the torque limiter. If the torque limiter is not used, breakage of the screwdriver may occur and could potentially harm the patient.

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Retract the sleeve when inserting the first screw with the aiming device. Advance the screw until the head of the screw contacts the plate.

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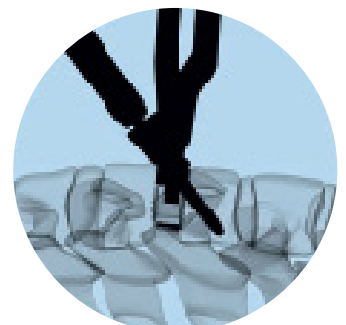
Use intraoperative images to verify the position of the screw.

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Do not tighten the screws until all the screws have been inserted.

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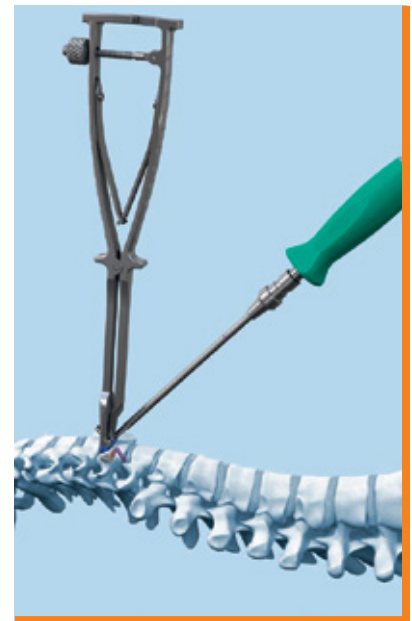
Select a drill bit with a suitable depth stop. Insert the bit into the handle and into the screw hole in the aiming device and drill until the depth stop of the drill bit contacts the guide.

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Use intraoperative images to verify the position of the drill bit. Remove the drill bit. Repeat the procedure for the remaining screw holes.

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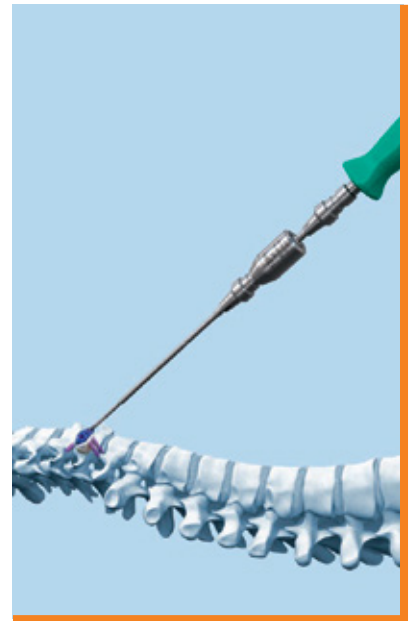
Remove the implant aiming device.

Load a screw selected onto the screwdriver with the torque limiter. The screw will be automatically fastened to the screwdriver. The holding sleeve can also be used to increase screw retention.

Advance the screw until the head of the screw contacts the plate.

Repeat the procedure for the remaining screw holes.

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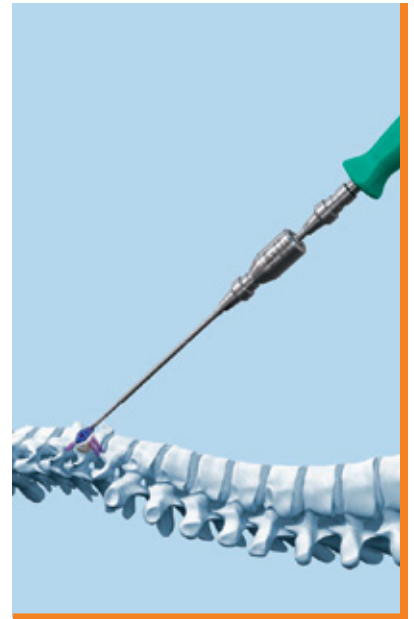
To lock the screw head in the plate, always use the torque limiter with the screwdriver to tighten each screw to the recommended 1.2 Nm torque.

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Fixation with drill guide and freehand screw

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If use of the aiming device is not the preferred surgical technique for the X-Zone cervical cage, follow this alternative technique.

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Select a drill bit with the appropriate length. Determine the entry point and screw path. The correct angulations for the screws are 40° in the caudal or cranial direction. The medial screws point 2.5° laterally; the lateral screws, 2.5° medially.

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Lateral screws should always point medially.

Insert the drill guide into the screw hole at the appropriate angle. The tip of the drill guide is designed to fit inside the screw hole of the plate and form the correct angle.

Insert the drill bit into the aiming device and drill until the drill bit stop contacts the aiming device.

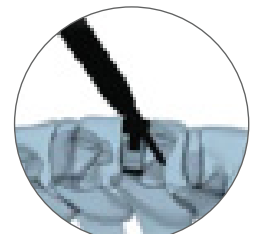
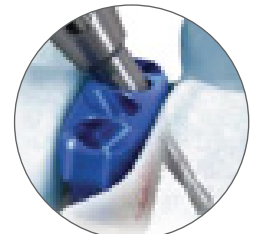
Remove the drill bit and the aiming device.

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*Load a screw onto the self-retaining screwdriver with the torque limiter. The screw will be automatically fastened to the screwdriver. The holding sleeve can also be used to increase screw retention.
Advance the screw until the head of the screw contacts the plate.*

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Repeat steps B1 and B2 for the remaining screws.

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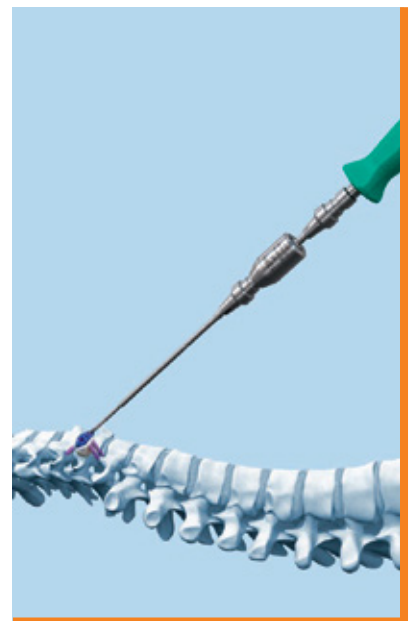
If the torque limiter is not used, breakage of the screwdriver may occur, which could increase patient risk.

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The screws placed with the surgical technique may not always be flush with the plate, although they will be sufficiently locked when 1.2 Nm torque is reached.

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Fixation with the threaded tip drill bit guide and freehand screw

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Determine the path of the threaded drill guide. The correct angulations are 40 ° in the caudal or cranial direction. The medial screws point 2.5 ° laterally; the lateral screws, 2.5 ° medially.

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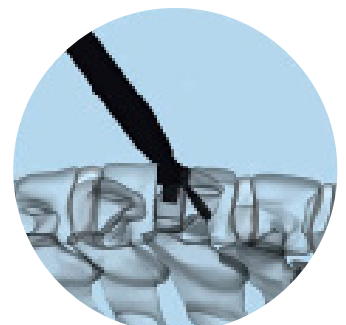
Retire la broca y la guía roscada.

Insert the threaded tip drill guide into the screw hole at the appropriate angle. The tip of the drill guide is designed to fit inside the screw hole of the intersomatic plate and guide the correct angle.

Select a drill bit with the appropriate length. Insert the drill bit into the aiming device and drill until the drill bit stop contacts the aiming device.

Remove the drill bit and the threaded guide.

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Use intraoperative images to verify the position of the drill bit.

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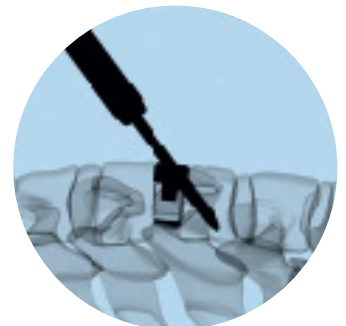
Load a screw onto the screwdriver with the torque limiter. The screw will be automatically fastened to the screwdriver. The holding sleeve can also be used to increase screw retention.

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Repeat steps C1 and C2 for the remaining screws.

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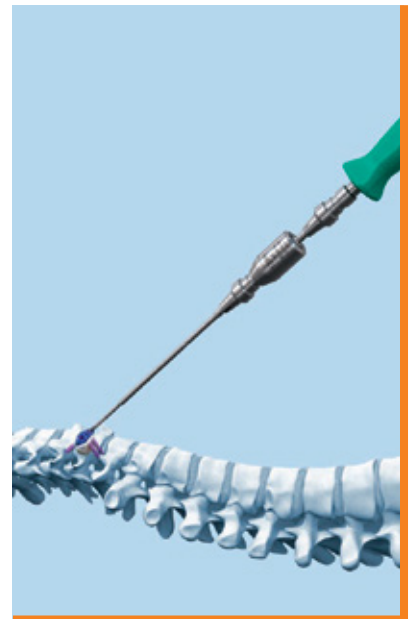
Fixation with awl and freehand screw

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If the surgeon prefers to use the awl instead of the drilling technique, this alternative procedure can be used.

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Determine the entry point and screw path. The correct angulations for the screws are 40 ° in the caudal or cranial direction. The medial screws point 2.5 ° laterally; the lateral screws, 2.5 ° medially.

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Lateral screws should always point medially. Insert the awl at the appropriate angle into a screw hole in the plate and push down, while twisting the handle. Remove the awl, maintaining the alignment of the hole and plate.

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Use intraoperative images to verify the position of the awl.

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The tip of the awl is designed to fit inside the screw hole of the plate and form the correct angle.

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Make sure the awl does not move the implant in relation to the vertebral body. For particularly hard bones, drilling is recommended to minimize implant movement.

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*Load a screw onto the screwdriver with the torque limiter. The screw will be automatically fastened to the screwdriver. The holding sleeve can also be used to increase screw retention.
Advance the screw until the head of the screw contacts the plate.*

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Repeat steps D1 and D2 for the remaining screws.

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The screws placed with the surgical technique may not always be flush with the plate, although they will be sufficiently locked when f 1.2 Nm torque is reached.

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If straight instruments cannot be used due to the anatomical features of the patient, the awl and the angulated screwdriver can be used.

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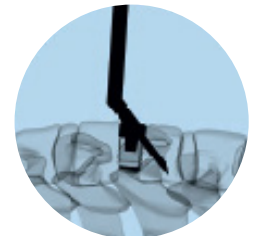
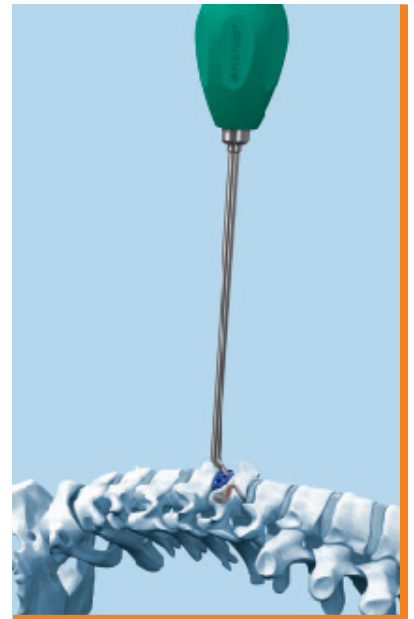
Insert the awl at the appropriate angle into the screw hole of the plate and tap with the mallet until the awl is properly seated.

Remove the awl, maintaining the alignment of the hole and plate.

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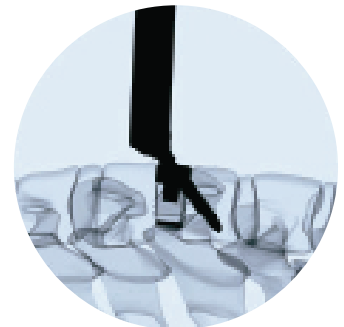
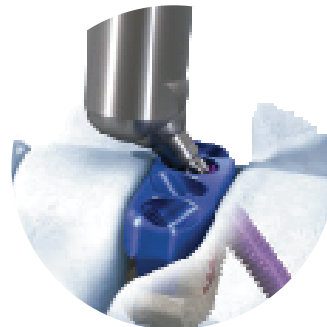
*Select the screw with the appropriate length according to the preoperative planning and intraoperative results.
Insert the screw in the angled screwdriver. Advance the screw until the head of the screw contacts the plate.*

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Repeat steps E1 and E2 for the remaining screws.

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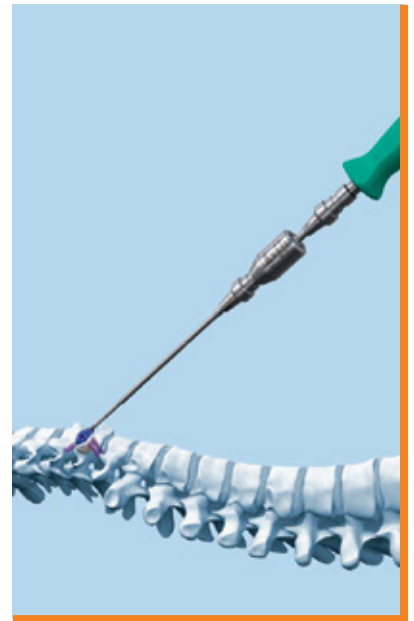
To lock the screw head in the cage, always use the torque limiter with the straight screwdriver or the optional angled locking screwdriver to tighten each screw to the recommended 1.2 Nm torque.

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The screws placed with the surgical technique may not always be flush with the cage, although they will be sufficiently locked when 1.2 Nm torque is reached.

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If it is necessary to remove the X-Zone locking cervical cage, apply the following technique.

Extraiga el tornillo

Drill the first pilot hole with the awl

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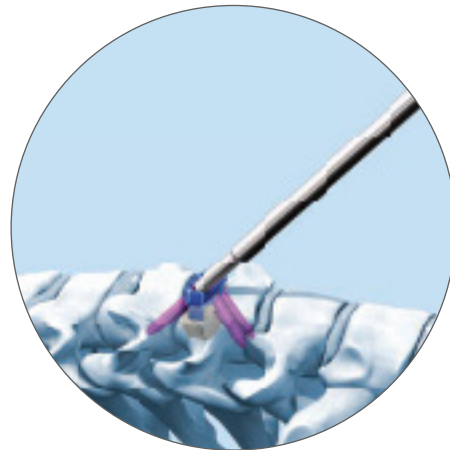
Attach the handle to the screwdriver shaft, then engage the assembled driver into the drive recess of the screw to be removed. Rotate the screwdriver counterclockwise to first loosen the screw from the X-Zone locked cervical cage. Continue to rotate the screwdriver counterclockwise to remove the loosened screw from the implant.

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After removing all the screws, the implant can be removed with the X-Zone locked cervical cage aiming device. Attach the guide to the implant by aligning the screw holes of the implant with the retention features on the aiming device and then expand the aiming device.

Once the implant is secure, remove it with care.

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It is recommended to use distraction in the disc space to facilitate removal.

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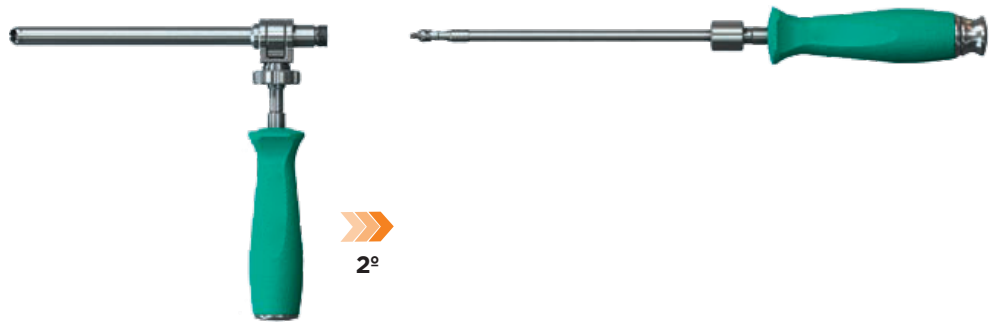
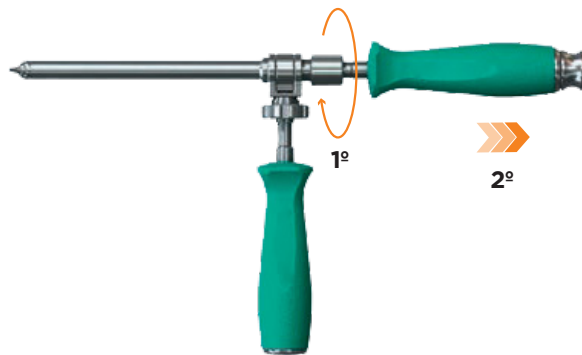
Instrument disassembly for washing

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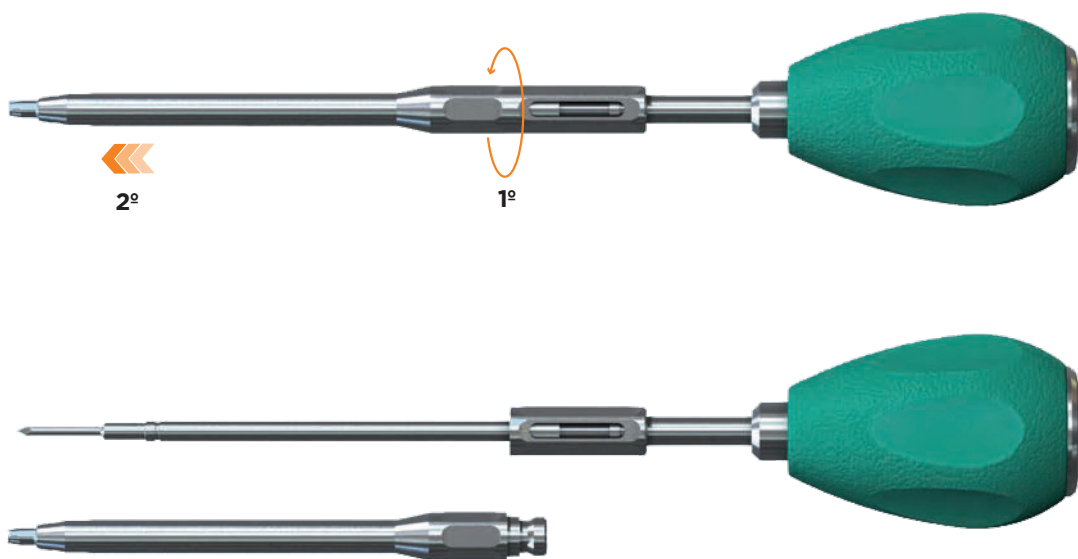
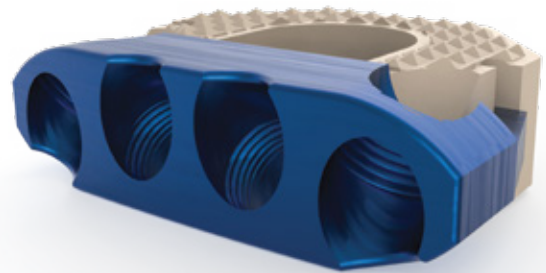
Screws for X-Zone locked surgical cage

TITANIO
TITANIUM

Código	Largo (mm.)	Color
1875-000-03	12	Bright Blue
1876-000-02	14	Bright Yellow
1877-000-13	16	Bright Magenta

INSTRUMENTOS

INSTRUMENTS

Código	Denominación	Cant.
78802-000-35	551001 ATORNILLADOR ANGULADO TORX T8 <i>551001 TORX T8 ANGLED SCREWDRIVER</i>	1
78803-000-35	551002 MARTILLO <i>551002 MALLET</i>	1
78804-000-35	551003 IMPACTOR DE ESPONJOSA <i>551003 CANCELLOUS BONE IMPACTOR</i>	1
78805-000-35	551004 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 5 MM X-ZONE <i>551004 TRIAL SPACER FOR 5 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78806-000-35	551005 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 6 MM X-ZONE <i>551005 TRIAL SPACER FOR 6 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78807-000-35	551006 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 7 MM X-ZONE <i>551006 TRIAL SPACER FOR 7 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78808-000-35	551007 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 8 MM X-ZONE <i>551007 TRIAL SPACER FOR 8 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78809-000-35	551008 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 9 MM X-ZONE <i>551008 TRIAL SPACER FOR 9 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78810-000-35	551009 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 10 MM X-ZONE <i>551009 TRIAL SPACER FOR 10 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78811-000-35	551010 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 11 MM X-ZONE <i>551010 TRIAL SPACER FOR 11 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78812-000-35	551011 IMPLANTE DE PRUEBA PARA CAGE CERVICAL BLOQUEADO 12 MM X-ZONE <i>551011 TRIAL SPACER FOR 12 MM X-ZONE LOCKED SURGICAL CAGE</i>	1
78813-000-35	551012 IMPACTOR CON PUNTA ESFÉRICA <i>551012 IMPACTOR WITH BALL TIP</i>	1
78814-000-35	551013 IMPACTOR PLANO <i>551013 FLAT IMPACTOR</i>	1
78815-000-35	551014 GUÍA DE BROCA CON MANGO <i>551014 DRILL GUIDE WITH HANDLE</i>	1
78816-000-35	551015 VAINA DE SUJECIÓN PARA TORNILLO <i>551015 SCREW HOLDING SLEEVE</i>	1
78817-000-35	551016 BROCA CON TOPE DE 12 MM <i>551016 12 MM DRILL BIT WITH DEPTH STOP</i>	2
78818-000-35	551017 BROCA CON TOPE DE 14 MM <i>551017 14 MM DRILL BIT WITH DEPTH STOP</i>	2
78819-000-35	551018 BROCA CON TOPE DE 16 MM <i>551018 16 MM DRILL BIT WITH DEPTH STOP</i>	2
78820-000-35	551019 GUÍA DE BROCA CON PUNTA ROSCADA <i>551019 DRILL BIT WITH THREADED TIP</i>	1
78821-000-35	551020 ATORNILLADOR TORX T8 <i>551020 TORX T8 SCREWDRIVER</i>	2
78822-000-35	551021 PUNZÓN ANGULADO <i>551021 ANGLED AWL</i>	1
78823-000-35	551022 PUNZÓN CON VAINA <i>551022 SLEEVED AWL</i>	1
78824-000-35	551023 MANGO CON ACOPLE RÁPIDO <i>551023 QUICK CONNECT HANDLE</i>	2
78825-000-35	551024 MANGO CON ACOPLE PARA TORNILLO DE POSICIÓN <i>551024 HANDLE WITH COUPLING FOR POSITION SCREW</i>	1

INSTRUMENTOS

INSTRUMENTS

Código	Denominación	Cant.
78826-000-35	551025 PIEZA PARA COMPACTACION DE INJERTO PARA CAGE X-ZONE <i>551025 GRAFT COMPACTOR FOR X-ZONE CAGE</i>	1
78827-000-35	551026 SOPORTE DE MANGO PARA ATORNILLADOR ANGULADO TORX T8 <i>551026 HANDLE FOR TORX T8 ANGLED SCREWDRIVER</i>	1
78828-000-35	551027 PORTAIMPLANTES <i>551027 IMPLANT HOLDERS</i>	1
78829-000-35	551028 GUÍA PARA CAGE CERVICAL BLOQUEADO X-ZONE <i>551028 AIMING DEVICE FOR X-ZONE LOCKED SURGICAL CAGE</i>	1
78830-000-35	551029 TORNILLO DE POSICIÓN <i>551029 POSITION SCREW</i>	2
78831-000-35	551030 DISTRACTOR <i>551030 DISTRACTOR</i>	1
78832-000-35	551031 LIMITADOR DE TORQUE <i>551031 TORQUE LIMITER</i>	1

Productos autorizados por la ANMAT PM 2022-12
Medical device authorized by ANMAT PM-2022-12

DC-094-00